
venome®

Filler

Venome XXL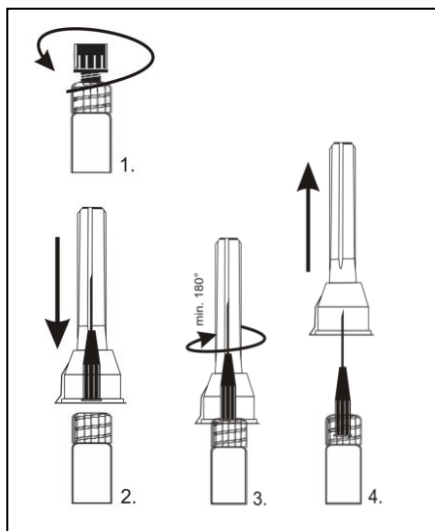

CE 0297

S & V Technologies GmbH
Neuendorferstraße 20a 16761 Hennigsdorf, Germany
Tel.: +49 3302-202-7900
Fax: +49 3302-202-7905
e-mail: info@skin-vision.eu

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Venome Filler Venome XXL

Medical device according to 93/42/EEC

Composition 1 ml contains 28 mg sodium hyaluronate.

Content Per sterile disposable syringe with Venome XXL, two sterile needles 27G1 / 2 for single use, for exclusive injection of Venome XXL; one Instruction for Use; as well as a set of labels for the patient record and to be handed over to the patient.

Indication Viscoaugmentation for correction of volume losses or defects related to disease or age in the face, e.g. for volume augmentation of cheeks and for correction of medium to deep wrinkles like marionette lines and nasolabial folds.

Note The intradermal injection of Venome XXL must only be performed by physicians, with appropriate training in dermal filler injection. The correct injection technique is essential for the success of the treatment. Injection into the wrong skin layer, an inappropriate technique or a high injection rate may lead to pronounced side effects.

Properties Venome XXL is a monophasic, sterile, isotonic, apyrogenic clear gel made of highly purified crosslinked hyaluronic acid of biofermentative origin and thus free of animal traces. Hyaluronic acid, a natural polysaccharide, is present as an important structural element in skin, in subcutaneous and connective tissue, in synovial tissue as well as in synovial fluid, and it belongs to the few substances that are chemically identical in all living organisms.

Mode of action The injection of Venome XXL into the extracellular matrix adds volume to the tissue. Wrinkles, creases and folds are smoothed. The effect may last at least 6 - 9 months; however, it is influenced by the depth of the treated wrinkles, the technique of injection, as well as biological, lifestyle and environmental factors. Degradation of crosslinked hyaluronic acid occurs enzymatically through the endogenous enzyme hyaluronidase as well as by oxidative hydroxylation.

Application and Techniques Venome XXL should be used at room temperature. It is suitable for the replacement or augmentation of tissue volume in order to compensate the volume loss caused by skin aging. It is intended for mid-to-deep dermal layer injection. The volume to be injected per treated wrinkle differs individually and depends on the grade of the wrinkle. It is recommended to slightly undercorrect the wrinkle. Also in the case of wrinkles that have already been corrected once, consider the risk of overcorrection. Selection of the injection technique depends on the type and degree of the wrinkles/folds. Markedly hardened folds that do not disappear by stretching will be difficult to lift up. In order to prevent the formation of nodules, an even distribution of the gel should be ensured. Slow injection will help prevent or lessen injection related side effects. Following the injection, the gel should be distributed in the tissue by gentle massage. It is recommended to cool the treated areas of the face for about 10 minutes following injection. After the initial treatment, touch-up injections may be necessary in some cases to achieve the desired correction of the wrinkles.

Contraindications Venome XXL is contraindicated in case of:
• allergy to hyaluronic acid
• auto-immune disease or during immunotherapy
• acute skin infection in the area to be treated (e.g. acne)
• acute influenza and feverish cold
• acute systemic infection (herpes, psoriasis)
• coagulation disorder
• a tendency to form scars
• pregnancy or breast-feeding
• people under age of 18

Venome XXL must not be injected into:
• blood vessels
• orbita region, glabella and forehead
• joints
• breasts

Precautions As a rule, the physician should carefully investigate the patient history with respect to diseases, other anti-aging treatments and individual factors that might increase the risk of adverse reactions. The patient should be informed prior to treatment about the concept of tissue augmentation with hyaluronic acid, common injection-related reactions and possible side effects. Venome XXL should only be injected into healthy, non-inflamed and disinfected areas of skin. It is recommended to pretreat the injection area with an anaesthetic cream, patch or local block. Do not overcorrect. Excessive hyaluronic acid gel implantation may result in formation of nodules, large swellings and dislocation. When using a sharp needle for gel implantation the fine blood vessels may be injured, inducing a hematoma. The inappropriate use of a blunt cannula may increase the trauma in the skin tissue, resulting in a stronger swelling. Patients should not take aspirin, ginkgo preparations or other blood thinning substances one week prior to treatment. After treatment the patient should not expose to excessive sun or intense heat (solarium, sun bath, sauna) and extensive consumption of alcohol, at least until the initial reactions (redness, swelling) have resolved.

Side effects Immediately after treatment, injection related reactions like redness and swelling and in rare cases itching or bruising may be observed at the injection site(s). The day after a stronger swelling may occur but this will resolve spontaneously within 3-5 days. Allergic reactions are extremely rare, because Venome XXL is protein free due to the non-animal source. Rarely, formation of nodules or oedema may occur at the injection site. In case of an inflammatory reaction it is recommended that the patient visits the physician. In literature on filler materials based on hyaluronic acid, individual cases of granuloma, abscesses, delayed hypersensitivity reactions and discoloration have been reported. Furthermore in literature, other very rare individual cases of necrosis have been described, due to injection into arterial vessels or occlusion by the filler pressing against the arterial wall.

Warnings Venome XXL has been approved for intradermal implantation. It must not be mixed with other filler products (e.g. permanent tissue fillers). Venome XXL should not be combined with other methods of aesthetic medicine, such as peeling, dermabrasion or any form of treatment with waves (laser, radiofrequency, ultrasound) while the effects last (6 - 9 months). Venome XXL should not be used together with products that contain quaternary ammonium compounds because of possible incompatibilities. Before use, check the integrity of the packaging and the expiry date of the product. Do not use when packaging is damaged. If the 27G1/2 needle gets blocked, the injection should be interrupted and the needle should be replaced. For single use only. Discard any unused amounts. Re-Use may pose a risk of infection caused by contaminated sterile product. Do not resterilize. The packaging is not autoclavable. Do not use in the operating room.

Storage Between 2°C and 25°C, in a dry place, protected from sun light.

Disposal Directly after use dispose of syringe with unused gel and needles.

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