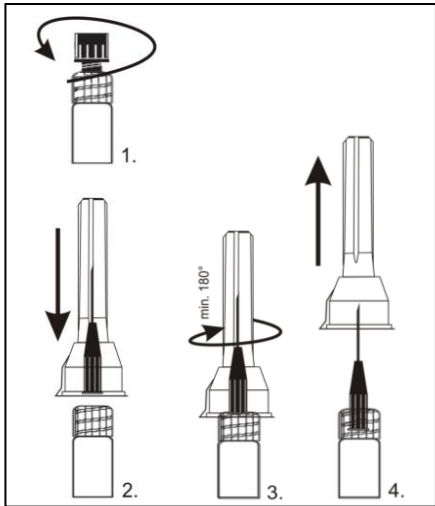
venome®

Filler

Venome S Lips Lidocaine

CE 0297

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Medical device according to 93/42/EEC

Description

Venome S Lips Lidocaine is a sterile, apyrogenic clear gel made of highly purified crosslinked hyaluronic acid in physiological buffer solution. It is of biofermentative origin and thus free of animal traces. 0.3% lidocaine has been added to the gel.

Composition

1 ml contains:

- sodium hyaluronate 14 mg
- lidocaine hydrochlorid 3 mg
- phosphate buffer 1 ml q.s.

Content

Per sterile disposable syringe with Venome S Lips Lidocaine two sterile needles 27G1/2 for single use, for exclusive injection of Venome S Lips Lidocaine; one Instruction for Use; as well as a set of labels for the patient record and to be handed over to the patient.

Indication

Viscoaugmentation for correction of volume losses or defects related to disease or age, e.g. for lighter volume augmentation in lips. Lidocaine has been added to the gel to reduce the local sensation of pain after the injection.

Mode of action

The injection of Venome S Lips Lidocaine into the lips adds volume to the tissue. This results in the desired volume augmentation. The effect may last between 5-6 months; however, it is influenced by the technique of injection, as well as biological, lifestyle and environmental factors. Crosslinking of the hyaluronic acid slows down degradation.

Application and Techniques

Intradermal injection of Venome S Lips Lidocaine should be performed by physicians, with appropriate training in dermal filler injection. The correct injection technique is essential for the success of the treatment. Injection into the wrong skin layer, an inappropriate technique or a high injection rate may lead to pronounced side effects.

Venome S Lips Lidocaine should be used at room temperature. It is a light gel which is suitable for light lip enhancement in order to compensate for volume loss caused by aging or to create additional volume. Volume depots are injected into the deep lip vermilion.

The volume to be injected per treated wrinkle differs individually. It is recommended to slightly undercorrect. Selection of the injection technique depends on the degree of the desired volume.

In order to prevent the formation of nodules, an even distribution of the gel should be ensured. Slow injection will help prevent or lessen injection related side effects.

Following the injection, the gel should be distributed in the tissue by gentle massage. It is recommended to cool the treated areas for about 10 minutes following injection. After the initial treatment, touch-up injections may be necessary in some cases to achieve the desired correction.

Contraindications

Venome S Lips Lidocaine is contraindicated in case of:

- allergy to hyaluronic acid, lidocaine hydrochloride or local anesthetics of the amide type
- auto-immune disease or during immunotherapy
- acute skin infection in the area to be treated (e.g. acne)
- acute influenza and feverish cold
- acute systemic infection (herpes, psoriasis)
- coagulation disorder
- a tendency to form scars
- pregnancy or breast-feeding
- people under the age of 18

Venome S Lips Lidocaine must not be injected into:

- blood vessels
- joints
- breasts

Precautions

As a rule, the physician should carefully investigate the patient's history with respect to diseases, other anti-aging treatments and individual factors that may increase the risk of adverse reactions. The patient should be informed prior to treatment about the concept of tissue augmentation with hyaluronic acid, common injection-related reactions and possible side effects.

For patients with antecedents, i.e. with an active autoimmune disease, or with a history of severe allergies, or after a recent anaphylactic shock, no clinical data are available on the tolerability and effectiveness of Venome S Lips Lidocaine.

The physician must evaluate each individual case to decide whether or not it is safe to inject Venome S Lips Lidocaine, depending on the type of disease and the associated treatment. For these patients, it is recommended to propose an initial double test. Refrain from injection if the disease is active or evolving. Furthermore it is recommended to carefully monitor these patients after the treatment.

Venome S Lips Lidocaine should only be injected into healthy, non-inflamed and disinfected areas of skin. The tissue shall not be overcorrected. Excessive hyaluronic acid gel implantation may result in formation of nodules, large swellings and dislocation.

When using a sharp needle for gel implantation the fine blood vessels may be injured, inducing a hematoma. The inappropriate use of a blunt cannula may increase the trauma in the skin tissue, resulting in stronger swelling.

Patients should not take aspirin, ginkgo preparations or blood thinning substances one week prior to treatment.

After treatment, the patient should not be exposed to excessive sun or intense heat (solarium, sun bath, sauna), at least until the initial reactions (redness, swelling) have resolved.

Warnings

Venome S Lips Lidocaine is only suitable for injection in lip tissue. It must not be mixed with other filler products (e.g. permanent tissue fillers). Venome S Lips Lidocaine should not be combined with other methods of aesthetic medicine, such as peeling, dermabrasion or any form of treatment with radiation (laser, radiofrequency, ultrasound) while the effects lasts (5-6 months). Venome S Lips Lidocaine should not be used together with products that contain quaternary ammonium compounds because of possible incompatibilities. Before use, check the integrity of the packaging and the expiry date of the product. Do not use when packaging is compromised. If the 27G1/2 needle gets blocked, the injection should be interrupted and the needle should be replaced.

For single use only. Discard any unused amounts. Re-use may pose a risk of infection caused by non-sterile product. Do not resterilize. The packaging is not autoclavable. Do not use in the operating room, because the outside of the syringe is not sterile.

Do not add other substances to Venome S Lips Lidocaine, and do not transfer the product from the syringe into another container.

General information on possible lidocaine side effects and restrictions for use

The total dose of administered lidocaine should be considered for concurrent treatments of different formulations (topical application). This also applies to simultaneous use with other local anaesthetics or substances with structural similarities to local anaesthetics of the amide type. The application may also result in a cumulative systemic toxicity.

The maximum dose per application of lidocaine HCl (without epinephrine) should be considered in case of a pretreatment with lidocaine. 300 mg (or 4.5 mg / kg) should not be exceeded in normal healthy adults. Overdose of lidocaine HCl regularly leads to symptoms in the central nervous system or of cardiovascular toxicity. During concurrent acidosis, hyperkalaemia, hypercalcaemia and hypoxia the toxic effects of local anaesthetics may be intensified and prolonged. Upon intoxication with local anaesthetics, centrally acting anaesthetics are contra-indicated.

The physician must carefully evaluate each case of epilepsy, heart disease, severe hepatic impairment, severe renal impairment or porphyria, in order to determine whether or not treatment with Venome S Lips Lidocaine can be performed safely. Hereby, the severity of the disease and the type of the associated treatment must be considered.

Special caution is required for patients with congenital methaemoglobinemia, glucose-6-phosphate dehydrogenase deficiencies and for patients taking methaemoglobin-inducing drugs.

For athletes it has to be noted that lidocaine treatment may cause a positive result in an anti-doping test.

Side effects

Immediately after treatment, injection related reactions like redness and swelling and in rare cases itching or bruising may be observed at the injection site(s). The day after a pronounced swelling may occur but this will resolve spontaneously within 3-7 days. Allergic reactions in the form of urticaria, oedema and broncho-spasm, anaphylactic reactions/shock (i.e. acute respiratory distress syndrome, circulatory reactions) are extremely rare, because Venome S Lips Lidocaine is protein free due to the non-animal source.

Rarely, formation of nodules or oedema may occur at the injection site.

In case of an inflammatory reaction it is recommended that the patient visits the physician. Literature on hyaluronic acid based filler materials reports individual cases of granuloma, abscesses, delayed hypersensitivity reactions and discolouration. Furthermore, literature also describes other very rare individual cases of necrosis due to injection into-arterial vessels or occlusion by the filler pressing against the arterial wall. Injection of lidocaine may cause a local redness or hypersensitivity reaction.

Storage

Between 2°C and 25°C, in a dry place and protected from sunlight.

Disposal

Directly after use dispose of syringe with unused gel and needles.

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