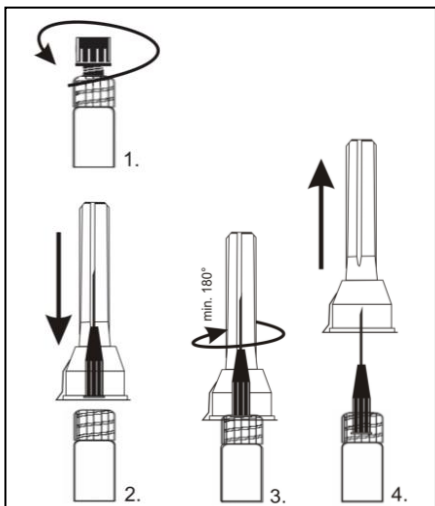
venome®

Filler

Venome M

CE 0297

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Table with multiple columns containing symbols and their corresponding instructions in various languages (German, English, French, Italian, Spanish, Polish, Czech, Slovak, Hungarian, Russian, Ukrainian, Chinese, Japanese, Korean, Arabic, Hebrew, Vietnamese, Thai, Indonesian, Malay, Tagalog, Hindi, Bengali, Urdu, Persian, Turkish, Greek, Latin, and others).

EN Instructions for Use
Venome Filler
Venome M

Medical device according to 93/42/EEC

Composition
1 ml contains 20 mg sodium hyaluronate.

Content
Per sterile disposable syringe with Venome M, two sterile needles
27G1/2 for single use, for exclusive injection of Venome M; one
instruction for use; as well as a set of labels for the patient record
and to be handed over to the patient.

Indication
Viscoaugmentation for correction of volume losses or defects
related to disease or age in facial skin, e.g. for correction of
medium wrinkles.

Note
Intradermal injection of Venome M must only be performed by
physicians with appropriate training in dermal filler injection. The
correct injection technique is essential for the success of the
treatment. Injection into the wrong skin layer, an inappropriate
technique or a high injection rate may lead to pronounced side
effects.

Properties
Venome M is a monophasic, sterile, isotonic, apyrogenic clear gel
made of highly purified crosslinked hyaluronic acid of

biofermentative origin and thus free of animal traces. Hyaluronic
acid, a natural polydisaccharide, is present as an important
structural element in skin, in subcutaneous and connective tissue,
in synovial tissue as well as in synovial fluid, and it belongs to the
few substances that are chemically identical in all living
organisms.

Mode of action
The injection of Venome M into the extracellular matrix adds
volume to the tissue. Wrinkles, creases and folds are smoothed.
The effect may last between 5-6 months; however, it is influenced
by the depth of the treated wrinkles, the technique of injection, as
well as biological, lifestyle and environmental factors.
Degradation of crosslinked hyaluronic acid occurs enzymatically
through the endogenous enzyme hyaluronidase as well as by
oxidative hydroxylation.

Application and Techniques
Venome M should be used at room temperature. It is suitable for
the replacement or augmentation of tissue volume in order to
compensate the volume loss caused by skin aging. It is intended
for upper to mid dermal layer injection.

The volume to be injected per treated wrinkle differs individually
and depends on the grade of the wrinkle. It is recommended to
slightly undercorrect the wrinkle. Also in the case of wrinkles that
have already been corrected once, consider the risk of
overcorrection. Selection of the injection technique depends on
the type and degree of the wrinkles/folds. Markedly hardened folds
that do not disappear by stretching will be difficult to lift up.
In order to prevent the formation of nodules, an even distribution
of the gel should be ensured. Slow injection will help prevent or
lessen injection related side effects.
Following the injection, the gel should be distributed in the tissue
by gentle massage. It is recommended to cool the treated areas
of the face for about 10 minutes following injection. After the initial
treatment, touch-up injections may be necessary in some cases
to achieve the desired correction of the wrinkles.

Contraindications
Venome M is contraindicated in case of:
• allergy to hyaluronic acid
• auto-immune disease or during immunotherapy
• acute skin infection in the area to be treated (e.g. acne)
• acute influenza and feverish cold
• acute systemic infection (herpes, psoriasis)
• coagulation disorder
• a tendency to form scars
• pregnancy or breast-feeding
• people under age of 18

Venome M must not be injected into:
• glabella
• blood vessels
• joints
• breasts

Not recommended for:
• treatment of orbita and front head lines

Precautions
As a rule, the physician should carefully investigate the patient
history with respect to diseases, other anti-aging treatments and
individual factors that might increase the risk of adverse reactions.
The patient should be informed prior to treatment about the
concept of tissue augmentation with hyaluronic acid, common
injection-related reactions and possible side effects.
Venome M should only be injected into healthy, non-inflamed and
disinfected areas of skin. It is recommended to pretreat the
injection area with an anaesthetic cream, patch or local block.
Wrinkles, creases and folds shall not be overcorrected. Excessive
hyaluronic acid gel implantation may result in formation of
nodules, large swellings and dislocation.
When using a sharp needle for gel implantation the fine blood
vessels may be injured, inducing a hematoma. The inappropriate
use of a blunt cannula may increase the trauma in the skin tissue,
resulting in a stronger swelling.
Patients should not take aspirin, ginkgo preparations or other
blood thinning substances one week prior to treatment.
After treatment the patient should not expose to excessive sun or
intense heat (solarium, sun bath, sauna), at least until the initial
reactions (redness, swelling) have resolved.

Side effects
Immediately after treatment, injection related reactions like
redness and swelling and in rare cases itching or bruising may be
observed at the injection site(s). The day after a pronounced
swelling may occur but this will resolve spontaneously within 3-5
days. Allergic reactions are extremely rare, because Venome M is
protein free due to the non-animal source.
Rarely, formation of nodules or oedema may occur at the injection
site.
In case of an inflammatory reaction it is recommended that the
patient visits the physician.
In the literature for filler materials based on hyaluronic acid,
individual cases of granuloma, abscesses, delayed
hypersensitivity reactions and discoloration have been reported.
Furthermore in literature, other very rare individual cases of
necrosis have been described, due to injection into arterial vessels
or occlusion by the filler pressing against the arterial wall.

Warnings
Venome M is only suitable for intradermal implantation. It must not
be mixed with other filler products (e.g. permanent tissue fillers).
Venome M should not be combined with other methods of
aesthetic medicine, such as peeling, dermabrasion or any form of
treatment with waves (laser, radiofrequency, ultrasound) while the
effects lasts (5-6 months). Venome M should not be used together
with products that contain quaternary ammonium compounds
because of possible incompatibilities. Before use, check the
integrity of the packaging and the expiry date of the product. Do
not use when packaging is compromised. If the 27G1/2 needle
gets blocked, the injection should be interrupted and the needle
should be replaced.
For single use only. Discard any unused amounts. Re-use may
pose a risk of infection caused by sterile product. Do not
resterilize. The packaging is not autoclavable. Do not use in the
operating room.

Storage
Between 2°C and 25°C, in a dry place protected from sun light.

Disposal
Directly after use dispose of syringe with unused gel and needles.

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