

CONTRAINDICATIONS FOR USE

The device cannot be used in patients who are hypersensitive to any of the device's components. The product cannot be used on damaged and/or skin with a significant break in the continuity of the skin. The device cannot be used if this type of treatment support is a contraindication or is insufficient.

ATTENTION

The product cannot be used in children under 3 years of age, in pregnant women and breastfeeding, without prior consultation with a specialist doctor.

Due to the presence of concentrated natural active ingredients, during the first contact with the skin, temporary redness may occur, which should subside after several minutes.

If the redness does not disappear or subside, do not use the product and consult a specialist. Before the first application of the product, it is recommended to check for irritation on the forearm or arm.

WARNINGS

The product is intended for use on intact skin, for external use only. To avoid the risk of incorrect use, the product should be applied to small children only under the supervision of adults.

Do not swallow.

Avoid contact with eyes.

In case of contact with the eyes, rinse with plenty of water.

Do not use after the expiry date stated on the packaging.

To ensure effective operation, the product should be used within 6 months from the first opening of the unit package. After this date, the properties of the product may deteriorate significantly.

Use the product in accordance with the intended use and in accordance with the instructions provided in this manual.

SIDE EFFECTS

During laboratory and dermatological tests, no irritating or sensitizing effect was found on the skin. In case of side effects, irritation or sensitization, stop using the product and consult a doctor.