

Contraindications

Should not be used:

- For injections other than intradermal,
- For correction of superficial wrinkles,
- In the orbital area (eyelids, wrinkles and bags under the eyes),
- Into the bloodstream (intravascular),
- Where any non-absorbable filler implant has already been applied,
- Do not perform readjustments.

The use of the product is not recommended:

- In patients with known hypersensitivity to hyaluronic acid, lidocaine or amide-type local anesthetics,
- In patients with a history of autoimmune diseases or undergoing immunotherapy
- In patients suffering from epilepsy that is not controlled with medication.
- In patients suffering from porphyria.
- In patients prone to hypertrophic scar formation.
- In patients suffering from a history of recurrent angina or acute rheumatoid arthritis with a cardiac localization,
- In pregnant or lactating women,
- In children,
- In areas with inflammatory and/or infectious skin conditions (acne, rashes),
- In direct combination with laser therapy, deep chemical peeling or dermabrasion.