

Composition

- Cross-linked hyaluronic acid 25mg
- Lidocaine 3mg
- Phosphate buffer pH 7.2

Contraindications

Should not be used:

- For injections other than intradermal,
- For the correction of superficial wrinkles,
- Around the eye sockets (eyelids, wrinkles and bags under the eyes),
- Into the bloodstream (intravascularly),
- In a place where any non-absorbable filling implant has already been used,
- Do not re-adjust.

It is not recommended to use the preparation:

- In patients with known hypersensitivity to hyaluronic acid, lidocaine or local anesthetics of the amide type,
- In patients with a history of autoimmune diseases or undergoing immunotherapy
- In patients suffering from epilepsy that is not controlled with medication.
- In patients suffering from porphyria.
- In patients prone to hypertrophic scarring.
- In patients suffering from a history of recurrent angina or acute rheumatoid arthritis with cardiac localization,
- In pregnant or breastfeeding women,
- In children,
- In places where skin inflammation and/or infection occurs (acne, rash),
- In direct combination with laser therapy, deep chemical peeling or dermabrasion.