

### **Composition**

- Cross-linked hyaluronic acid 25mg
- Lidocaine hydrochloride 3mg
- Phosphate buffer pH 7.2

### **Contraindications**

The product should not be used:

- for other than intradermal injection,
- for correction of superficial wrinkles,
- in the area of the eye sockets (eyelids, wrinkles and bags under the eyes),
- into the bloodstream (intravascular),
- if a non-absorbable filler implant has already been used,
- do not make recorections.

### **The use of the preparation is not recommended:**

- In patients with known hypersensitivity to hyaluronic acid,
- in patients with a history of autoimmune diseases or who are receiving immunotherapy,
- in patients with epilepsy that is not controlled with medication,
- in patients with porphyria,
- patients with a tendency to hypotrophic scars,
- in patients with recurrent angina in history or acute rheumatoid arthritis with cardiac localization,
- in pregnant or breastfeeding women,
- in children,
- in areas with skin inflammations and/or infections (acne, herpes),
- in combination with peelings, laser therapy or dermabrasion.