

## Research

Safety through clinical studies, which first led to the acquisition of two patents and then to CE certification, which is required to place the active ingredient on the European market.

Stimulation has been shown to induce a high degree of controlled inflammation.

Photographs of histopathological sections show an orderly distribution of molecules in the skin, surrounded by freshly produced collagen fibers.

## Patents

- Patent: No. 10-1481441 Manufacturing method of polymeric microparticles having network structure - Method for the production of cross-linked polymeric microparticles.

- Patent: No. 10-1725279 A process for the preparation of polylactic acid microparticles by a spray method - Spray method for the preparation of polylactic acid microparticles - spherical, porous form.

## Contraindications

- Pregnancy
- Breastfeeding
- Active infection
- Collagen diseases
- Active, unregulated autoimmune diseases
- Allergy to ingredients of the preparation
- Inflammation of the skin in the area of the planned treatment

## Warnings

Juvelook should not be used around the eye area (eyebrows, corners of the eyes, eyelashes, upper eyelids and tear troughs). Important warnings: Serious side effects, such as loss of vision, may occur if the product is injected into blood vessels. Therefore, it should not be used in the eye area (eyebrows, eye area, eyelashes, upper eyelid, tear duct). The product should not be injected into the superficial skin layer in the eye area (eyebrows, eye area, upper eyelid, tear duct) in order to avoid the formation of nodules or lumps. The formation of such nodules or lumps can be caused by improper procedures (e.g. injection into the skin surface, excessive amount of injections).

Massaging the injection site to distribute the product evenly may reduce the appearance of nodules and lumps. General precautions: Avoid injections into thinner skin areas such as the eye area as there is a high risk of nodules or lumps forming.