

Contraindications

- Do not inject into the eye area (eyelid, lower eyelid, crow's feet) or glabella.
- Do not inject intravascularly.
- Injection into blood vessels may cause embolism, vascular occlusion, ischemia, or infarction.
- Do not make excessive corrections.

Do not use in

- Patients with untreated epilepsy
- Patients with a tendency to develop hypertrophic scars
- Patients with known hypersensitivity to hyaluronic acid
- Patients with hypersensitivity to lidocaine or other local amide anesthetics
- Patients with porphyria
- Pregnant or breastfeeding women
- Children
- Skin areas with problems such as inflammation or infection (acne, herpes, etc.)
- Directly after laser therapy, deep chemical peels, or dermabrasion
- Injection is not recommended for superficial peels if this leads to a severe inflammatory reaction