

Contraindications

The use of Hydralix Gentle/Deep/ Ultra Deep/ Lips products is contraindicated

- In patients with known sensitivity to any of the product's ingredients,
- In patients suffering from skin diseases or with an unusual skin condition,
- In patients suffering from infection or inflammation (both acute and chronic) at or near the surgical site,
- In patients prone to scar formation, hypertrophic scars or developing inflammatory skin conditions,
- In patients under 18 years of age,
- In breastfeeding or pregnant women,
- In patients with known hypersensitivity to steroids or in which steroid treatment is contraindicated,
- For injection into the epidermis,
- For injection into surgical sites,
- In patients presenting with herpes zoster.

Hydralix Gentle is also contraindicated for:

Injection into the eyelids,
The product can only be injected into the bags under the eyes by well-trained physicians who are well versed in the physiology of the treatment site

Hydralix Deep is also contraindicated for:

Injection into the orbital area (e.g., eyelids, bags under the eyes, wrinkles near the eyes),
Injections into the smooth forehead and area around the eye,
Injections into the lips and area around the mouth,
Injections into the superficial layer of the dermis.

Hydralix Ultra Deep is contraindicated for:

Injections in the orbital area (e.g., into the eyelids, bags under the eyes, wrinkles near the eyes),
Injections into the smooth forehead and area around the eye,
Injections into the lips and area around the mouth,
Injections into the superficial and middle layers of the dermis and surface wrinkles.

Hydralix Lips is also contraindicated for:

Injection into the orbital area (e.g., into the eyelids, bags under the eyes, wrinkles near the eyes),
Injection into the superficial layer of the dermis.