

Recommendations

Do not use if the vial has been damaged in any way.

The contents of the vial must be used immediately after opening.

Store at 2°C-30°C.

The product is intended for use by a professional.

Contraindications/side effects

Contains collagen of animal origin from a porcine source.

However, patients who are found to be hypersensitive to any of the product's ingredients should be tested by topical injection into the arm and observed for 1 hour before use.

Warnings and precautions

Slight redness in the injection area may be caused by the mechanical effect of the needle insertion or by a skin reaction. During application, burning/pain may occur at the injection site, which normally subsides spontaneously within 5-10 minutes after treatment.

Skin cleansing/disinfection is required before and after administration. Abscess-forming bacteria can lead to the formation of an abscess in the injection area.

KEEP OUT OF THE REACH OF SMALL CHILDREN.

Do not use after the expiry date. The expiry date refers to products that are properly stored in the undamaged original packaging. The product must be used immediately after opening.

Instructions for use

MD-Muscle can be used alone or in combination with other medical products from the same range to customize the treatment according to the clinical situation.

The product can be used in patients who require collagen supplementation or topical anti-wrinkle therapy.