
ADVERSE SIDE EFFECTS:

Unknown. The product is hypoallergenic. No adverse effects have been reported.

The patient should be informed of the existence of potential undesirable side effects associated with the product, which may occur immediately or after a sometime. To minimise the risk of side effects, in first seek the advice of a highly qualified and competent plastic surgeons/doctors. The incidence of undesired side effects such as minor haematomas, bruising, swelling, infection, redness or minor bleeding after treatment with hyaluronic acid is comparable to other fillers. Some patients also complain of a feeling of tension and warmth in the injection area. Such secondary symptoms can be alleviated, applying ice to the area of application. They usually disappear within a short period of time. The occurrence of undesirable side effects more serious than those mentioned above, is unlikely. However, if any of these occur, the patient should notify the physician immediately. The doctor must make sure that patients inform him or her of any undesirable side effects that will follow the procedure.

CONTRAINDICATIONS:

- DO NOT USE THE PRODUCT FOR APPLICATIONS OTHER THAN THOSE INDICATED IN THESE INSTRUCTIONS.
 - Do not inject MonoDermà Fillagen into the periorbital (eyelids, crow's feet) and plaque (forehead).
 - Do not inject into blood vessels (intravascular).
 - Do not overdo the corrective action.
 - Do not use in the case of known individual hypersensitivity to the product or any of its components.
 - Do not use in case of autoimmune connective tissue vascular disease (connective tissue disease)
- Not recommended for use during pregnancy and lactation, although there are no known any specific contraindications.
 - Do not inject into patients with untreated or ineffectively treated epilepsy;
 - Do not inject into patients with a tendency to hypertrophic scar formation or suffering from porphyria;
 - Do not use in children;
 - Do not use the product in direct combination with laser treatments, deep chemical peels or dermabrasion. In the case of superficial peeling
 - Do not inject the product if a strong inflammatory reaction has occurred.