

**The NOVUMA® preparation is contraindicated and should not be used in the following cases:**

- If the patient is allergic to any of the ingredients.
- If the patient has a history of anaphylactic shock.
- If the patient has previously had keloids or has a known predisposition to them.
- If there is a risk of impaired hemostasis in cases of bleeding that may occur due to injection in patients with diagnosed coagulation disorders.
- If the patient has an acute and/or chronic infection at the treatment site.
- If the patient has a history of herpes virus infection.
- If the patient is using systemic steroids, as they may impair connective tissue regeneration.
- If there are foreign bodies, such as liquids, silicone, or other solid particles, at the application site.
- In patients with diagnosed connective tissue diseases.
- If the patient has a systemic disease that delays wound healing or a condition that could cause tissue degradation at the implant site.