

Warnings

- Pluryal cannot be used in areas where there are breasts or current tendon, bone or muscle implants.
- After the procedure, until the swelling and redness are completely absorbed, the implantation sites must not be exposed to excessive heat (sun, UV tanning or laser) or intense cold.
- After use, needles and syringes should be disposed of in accordance with the procedures indicated for hospital waste.

Adverse reactions, warnings and contraindications

- As with any percutaneous injection, the following may occur: inflammatory reactions (redness, swelling, etc.), sometimes associated with itching and pain to touch, hematoma, induration or lumps at the injection site, discoloration or discoloration of the skin at the injection site.
- These reactions usually disappear within a few days. If they persist for more than a week, you should contact your doctor immediately.
- Pluryal is intended for intradermal injection and must not be injected into blood vessels. There are no reports of problems due to excessive doses or interactions with other medications. Do not use during pregnancy. Pluryal is available in sterile single-dose packages.
- The product cannot be reused for future applications or on other patients.
- Once opened, the product should be used immediately.
- Any product residues should be removed. Do not use the product if the packaging is damaged.
- Keep away from children. In rare cases, the product may cause allergic reactions.