

Composition/effect:

- High molecular weight sodium hyaluronate 1.5, obtained by bacterial fermentation, without any chemical modification.
- Other ingredients: Sodium chloride, disodium phosphate 2H₂O, monosodium phosphate 2H₂O, water

Special warnings:

The contents of the ampoule syringe are sterile. Each ampoule syringe is packed in a sealed blister. The outer part of the syringe is not sterile. Intradermal infiltration should only be performed by a physician. Although preclinical studies in laboratory animals indicate that the product has no potential toxicity to reproduction and fetal development, SEVENTY HYAL 2000 has never been tested in pregnant women.

Do not use if the packaging is damaged. Do not use after the expiry date indicated on the packaging.

The expiry date refers to the product stored in its original packaging, at a temperature not exceeding 25°C and protected from light.

SEVENTY HYAL 2000 is intended for single-dose administration and must be disposed of after use in accordance with local regulations.

Keep out of the reach and sight of children.

Side effects:

Local pain, itching, swelling or redness may occasionally occur at the injection site. These symptoms are usually mild and temporary.

To speed up recovery, it is advisable to apply ice to the injection site.

Interactions:

Disinfectants containing quaternary ammonium salts should not be used at the same time, as hyaluronic acid may precipitate in their presence.

SEVENTY HYAL 2000 can be used concomitantly with lidocaine-containing topical anesthetics.

Storage conditions:

Store at a temperature not exceeding 25°C and protect from light.