

Contraindications

- Must not be used in combination with peels, laser therapy, or dermabrasion,
- In areas where the skin is thin (e.g. on the forehead and around the eyes, including the eyelids, under the eyes, and for the correction of so-called crow's feet), in areas with visible vessels (e.g. glabella), and on the lips,
- In patients with a tendency to develop hypertrophic scars,
- In patients with known hypersensitivity to any of the ingredients of the product,
- In pregnant or breastfeeding women, as well as in children,
- On or near areas affected by inflammatory conditions or skin infections (acne, herpes, etc.).

